**LOT** DBCL23061A  
DBCL23062A

 2023-06-30

# MAS<sup>®</sup> Diabetes

## LIQUID ASSAYED DIABETES CONTROL

**IVD**

### INTENDED USE

**MAS<sup>®</sup> Diabetes** is intended for use as an assayed control material for monitoring Glycosylated Hemoglobin (A1c) assay procedures.

### PRODUCT DESCRIPTION

**Diabetes** is an assayed product prepared from human whole blood adjusted to specific concentrations of glycated hemoglobin. Preservatives and stabilizers are added to maintain product integrity.

**WARNING:** Diabetes assay contains ≤1% Methanol.

H371 - May cause damage to the central nervous system and eyes when ingested.

Do not breathe mist/vapors/spray. Wash hands thoroughly after handling. Do not eat, drink or smoke when using this product. If exposed or if you feel unwell: call a Poison Center or doctor/physician. Store locked up. Dispose of contents/container to location in accordance with local/regional/national/international regulations.

**CAUTION:** **Diabetes** is prepared from human source material. Components of the control which are derived from human source material have been tested using FDA accepted methods and found non-reactive for Hepatitis B Surface Antigen (HBsAg), Hepatitis C (HCV), HIV-1 and HIV-2. However, no test method can offer complete assurance that products derived from human source material are free of infectious agents. This control must be handled in accordance with recommendations from Centers for Disease Control/National Institutes of Health manual, "Biosafety in Microbiological and Biomedical Laboratories," 2009. The packaging of this product contains dry natural rubber.

### STORAGE AND STABILITY

**Diabetes** stored at -25 to -15°C is stable until the expiration date on the box. Once opened, vials of **Diabetes** are stable for 30 days when stored tightly capped at 2-8°C. Unopened vials of **Diabetes** are stable for 60 days from receipt when stored at 2-8°C. The control can be thawed and refrozen one time. **Self-defrosting freezers are not suitable.**

Bacterial contamination produces an increase in turbidity and/or a characteristic odor. Discard vial if evidence of microbial contamination is observed.

### CONTROL RANGES

The assigned ranges for these controls are based upon replicate assays of representative samples of the product by participating laboratories in accordance with established protocol. All values have been assigned with instruments and instrument manufacturer's reagents available at the time of assay. Subsequent instrument or reagent modifications may invalidate these assigned ranges.

Expected values may vary slightly with different reagent and/or methodologies used. Refer to the included table for values obtained for specific systems. Values listed are specific for this lot of control only. Good laboratory practice suggests that each laboratory establish its own parameters.

### INSTRUCTIONS FOR USE

Thaw control at room temperature (18-25°C) on a rocker or with periodic gentle inversion until liquid and then immediately store at 2-8°C. Thoroughly mix the contents of the vial before each use by gently inverting for several minutes. Open the vial and transfer the required quantity of control into a clean sample cup. Replace cap immediately and store the opened vial at 2-8°C. Assay controls in accordance with the reagent manufacturer's assay's and/or instrument's instructions for unknown specimens.

### QUALITY CONTROL

All quality control requirements should be performed in conformance with local, state and/or federal regulations or accreditation requirements.

### LIMITATIONS OF PROCEDURE

Values for each lot are given in the product insert. Different laboratories, however, may expect slightly different results. It is recommended that each laboratory establish its own expected range.

Accurate and reproducible results are dependent upon properly functioning instruments, reagents, and good laboratory technique. This product is intended for use as an assayed control for quantitative assays of listed constituents in human whole blood. This product is not intended for use as a calibrator. For professional use only.

### TECHNICAL ASSISTANCE

**In the USA**, for technical assistance, call 800-232-3342 or 510-979-5417. For insert updates and information, if your laboratory subscribes to LabLink xL, visit [www.maslablink.com](http://www.maslablink.com) and select LabLink Extra. Alternatively, to subscribe to LabLink xL call 800-232-3342 or 510-979-5451.

**Outside of the USA**, if your laboratory subscribes to LabLink xL, visit [www.maslablink.com](http://www.maslablink.com) select LabLink Extra. Alternatively, please contact your local sales office or authorized distributor.

**Cat. No.**  
DBCL-MP

**Description**  
MAS<sup>®</sup> Diabetes, Multi-Pack

**Size**  
6 x 1 mL

**LOT** DBCL23061A  
DBCL23062A

 2023-06-30

**thermo**  
scientific

For insert updates go to:  
[www.thermofisher.com/diagnostics](http://www.thermofisher.com/diagnostics)

# MAS<sup>®</sup> Diabetes

## LIQUID ASSAYED DIABETES CONTROL

**IVD**

	<b>CON</b>					<b>SI</b>				
	DBCL23061A		DBCL23062A		<b>U</b>	DBCL23061A		DBCL23062A		<b>U</b>
	$\bar{x}$	<R>	$\bar{x}$	<R>		$\bar{x}$	<R>	$\bar{x}$	<R>	
<b>A1c</b>										
Beckman AU Series										
Turbidimetric Immuno-inhibition (B00389)	**		**		g/dL	**		**		g/L
Beckman Synchron/DXC										
Immunoturbidimetric	0.46	0.37 - 0.55	0.93	0.74 - 1.12	g/dL	4.60	3.70 - 5.50	9.30	7.40 - 11.2	g/L
Ortho Clinical Vitros										
Turbidimetric Immuno-inhibition	**		**		g/dL	**		**		g/L
Roche Cobas c Systems										
TINIA (A1C-3)	0.54	0.43 - 0.65	1.12	0.90 - 1.34	g/dL	5.40	4.30 - 6.50	11.2	9.00 - 13.4	g/L
Thermo Fisher Scientific Indiko Systems										
Turbidimetric Immunoassay (Non U.S.)	**		**		g/dL	**		**		g/L

**LOT** DBCL23061A  
DBCL23062A

 2023-06-30

**thermo**  
scientific

For insert updates go to:  
[www.thermofisher.com/diagnostics](http://www.thermofisher.com/diagnostics)

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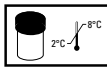
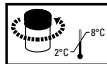
**If you have any questions concerning this notification, in the USA please contact the Technical Support Department at 800-232-3342 or 510-979-5417. Outside the USA, please contact your local Subsidiary or Distributor.**

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**USA**  
☎ 800-232-3342  
sales.diagnostics.fmt@thermofisher.com

**China**  
☎ +86 800-810-5118  
cdx.cn.info@thermofisher.com

**New Zealand**  
☎ 0800 933 966  
auinfo@thermofisher.com

**Switzerland & Austria**  
☎ +41 26 663 86 70  
cdx.ch.info@thermofisher.com

**Asia Pacific**  
☎ +61 1800 333 110  
cdd.asia.info@thermofisher.com

**France**  
☎ +33 1 40 86 65 20  
cdx.fr.info@thermofisher.com

**Nordic**  
☎ +47 2 325 0433  
info.nordic.cdd@thermofisher.com

**United Kingdom & Ireland**  
☎ +44 1442 868 940  
cdx.uk.info@thermofisher.com

**Australia**  
☎ +61 1800 333 110  
auinfo@thermofisher.com

**Germany**  
☎ +49 0800-40 40 771  
cdx.de.info@thermofisher.com

**South Africa**  
☎ +27117926790  
support-za.idd@thermofisher.com

**Canada**  
☎ 800-282-4075  
info.cddcanada@thermofisher.com

**Japan**  
☎ +81 (0)120-147-075  
JPYOK-CDD.QC@thermofisher.com

**Spain, Portugal & Italy**  
☎ +34 93589 8338  
cdx.es.info@thermofisher.com

**For countries not listed:**  
distributor.cdd@thermofisher.com

**Microgenics Corporation**  
46500 Kato Road  
Fremont, CA 94538 USA  
ISO 13485 Certified Company  
www.thermofisher.com/diagnostics

**B.R.A.H.M.S GmbH**  
Neuendorfstrasse 25  
16761 Hennigsdorf, Germany

DBCL-INS-VA  
Rev. 17 2020 12