

ONE STEP IMMUNOLOGICAL FECAL OCCULT BLOOD TEST



10
MINUTES
RESULT

DON'T LET POTENTIAL GI DISORDERS GO UNNOTICED

This rapid, immunochemical device is designed to provide accurate results right in the comfort of laboratories or physician's offices. It is useful in determining gastrointestinal (GI) bleeding in a number of GI disorders, e.g. diverticulitis, colitis, polyps, and colorectal cancer.

No more complex procedures or lengthy waiting times - get accurate results within minutes. Its efficiency saves time and resources, enabling healthcare professionals to focus on what truly matters - delivering exceptional patient care. The advanced immunochemical technology ensures that even minuscule amounts of occult blood are detected with precision, leaving no room for error. By using this test, healthcare providers can confidently identify GI bleeding and initiate prompt treatment plans accordingly.

SENSITIVITY

levels of human hemoglobin in feces:
as low as **0.2 ugHb/ml.**

SPECIFICITY

human hemoglobin

MDA APPROVED



Certificate No.: IVDB8486423-146332

PRODUCT INFORMATION

ITEM	QUANTITY	STORAGE & STABILITY
Hemosure One Step Immunological Fecal Occult Blood Test	30 Tests/ kit	2°C – 30°C shelf life: 23 months

HOW TO USE



COMPONENTS

Individually packed test devices	30pcs
Extraction buffer (2ml)	30pcs
Package insert	1pc

1 stick at **6** different sites



Loosen Fecal Collection Tube cap.

Randomly insert the Applicator Stick into the fecal sample.

2



Close and tighten the cap thoroughly.

Shake the tube well to mix the sample with the Extraction Buffer.

3 dispense **3** drops



Open the tip cover and squeeze tube to dispense 3 drops onto the sample area of the strip.

4



5-10 MINUTES

Start timer and read result within five (5) to ten (10) minutes.

DO NOT INTERPRET RESULT AFTER TEN (10) MINUTES.

RESULT INTERPRETATION

POSITIVE



One band appearing in the "C" region, the other in the "T" region.

NEGATIVE



Only one color band appearing in the "C" region.

INVALID



Control line fails to appear.

ORDERING INFORMATION

CAT. NO	ITEM	QUANTITY
T1-CK30T	Hemosure One Step Immunological Fecal Occult Blood Test	30 Tests / kit

DAIICHI BIOTECH SERVICES SDN BHD (925272-K)

Address: B-SG-20, Sunway Geo Avenue, Jalan Lagoon Selatan, Bandar Sunway, 47500 Subang Jaya, Selangor.

Tel: +603-5021 4600 / 4677 **Fax:** +603-5021 4668 **E-mail:** inquiry@daiichibiotech.com.my

